



AESTHETICS

LipoLas[®] – LaserLift



Technology
meets Anatomy



Naturally
firm skin –
gentle, precise
and without
surgical
intervention

- Effective skin tightening and fat reduction without surgery
- Minimally invasive, outpatient and efficient
- Expanding your aesthetic treatment spectrum
- High demand with short recovery time



**Schedule training
dates now!**

High-Precision Laser Technology

LipoLas® – LaserLift is a modern, minimally invasive treatment designed to reduce fat deposits whilst also tightening the skin. Using an ultra-fine laser fibre, both the deeper tissue and the superficial skin layers are treated simultaneously.


The result is visibly firmer contours and a natural improvement in skin quality – without the need for surgery.

How Does LaserLift Work?

LaserLift uses advanced laser technology to deliver precise energy beneath the skin. The laser energy is selectively absorbed by water, fat, and haemoglobin within the treated tissue.

This process gently breaks down fat cells, seals small blood vessels to minimise bleeding and bruising whilst also stimulating the production of new collagen fibres. This increased collagen helps tighten the skin, improve its elasticity and supports the body's natural healing process.

The treatment offers targeted body contouring with minimal downtime providing natural-looking results and progressive skin tightening as the collagen production continues over the following weeks and months.

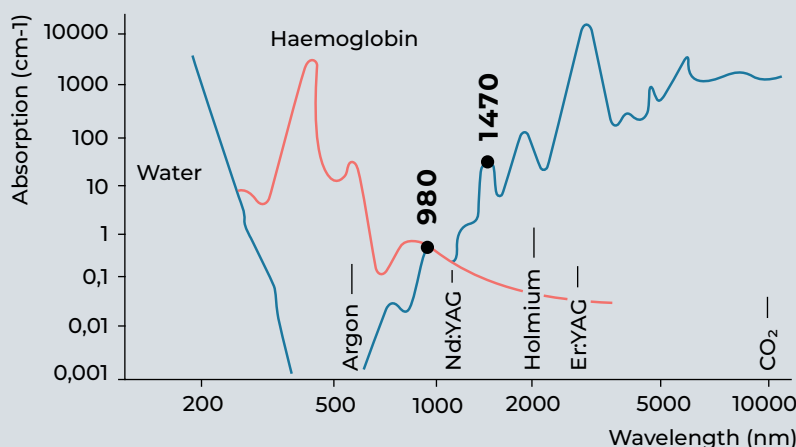

Laser
absorption in
haemoglobin
& water

Offers our Patients the Perfect Balance of Contour and Tightening

LipoLas® is ideal for patients who wish to reduce unwanted fat whilst also improving skin firmness. It is particularly well suited for those experiencing less skin elasticity following weight loss, pregnancy, or previous liposuction.

Its unique dual approach to fat reduction and skin tightening provides comprehensive body contouring in a single minimally invasive treatment, delivering natural-looking results with minimal downtime.



AFTER WEIGHT LOSS

For tightening sagging skin and improving body contour.



FOR AGE-RELATED SKIN LAXITY

For a visibly smoother, more youthful appearance.



AFTER LIPOSUCTION

For targeted follow-up treatment and further optimisation of results.



**Video: Thigh rejuvenation
with LipoLas® LaserLift**



Triple Action for Visible Results

1

FAT REDUCTION

Targeted treatment of small fat deposits, even in sensitive areas.

2

SKIN TIGHTENING

Activated collagen production ensures a smoother, firmer skin appearance.

3

LONG-TERM REGENERATION

Natural collagen regeneration sustainably improves skin structure.

„LipoLas LaserLift suits a wide range of indications. This simple and safe procedure works for both larger combined and smaller outpatient treatments ensuring high patient satisfaction.“

Dr. Nicole Caroline Haas

Plastic Surgery, Munich



Versatile – for Face and Body



Face



Chin / Neck



Eye



Upper Arm



Knee



Thigh




Buttocks



Abdomen

YOUR BENEFITS

- Minimally invasive - no visible scars
- No anaesthesia is usually required
- Fast recovery with minimal downtime
- Precise and gentle application
- Long-lasting, natural results

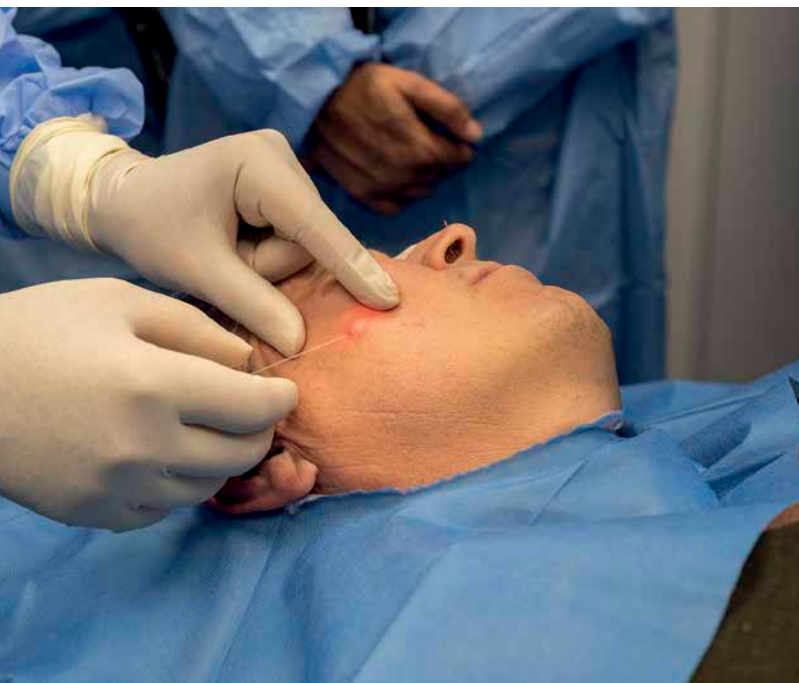

LipoLas® – LaserLift

represents a new generation of aesthetic laser treatment: effective, gentle and versatile, for patients who value natural results and modern treatment methods.

The Most Comprehensive Laser Portfolio for Aesthetics



**LEONARDO®
Mini 1470**

**LEONARDO®
DUAL 45**



Technical Data

Model	LEONARDO® Mini 1470 Set	LEONARDO® DUAL 45 Set
REF	SL1470nm12W	SL980+1470nm45W
Wavelength	12 W (1470 nm)	max. 45 W (1470 nm / 15 W + 980 nm / 30 W), separately adjustable
Fiber Diameter	≥ 360 µm	≥ 360 µm
Pilot Beam	635 nm, max. 4 mW	532 nm and 635 nm, green 1 mW, red 4 mW, user-defined intensity
Treatment Mode	CW, Pulse Mode (optional), ELVeS® Signal	CW, Pulse Mode, Derma Mode
Pulse Duration/Pause	0.01 to 180 s / 0.01 to 180 s	0.01 to CW / 0,01 to 60 s
Power Supply	110 to 240 VAC, 50 to 60 Hz (12 VDC @ 100 W)	110 to 240 VAC, 50/60 Hz, 450 VA
Batteries	Li-ion Batteries	–
Dimensions (H×W×D)	6 cm × 9 cm × 21.5 cm	28 cm × 37 cm × 9 cm
Weight	900 g	8.5 kg

All the laser sets described above (REF No.) include 3 pairs of safety goggles, foot switch, interlock connector, power cord and manual in a transport case

Fibers

REF	Product	PU*	Lenght [m]	AD ø [mm]
503200292	Bare Fiber 365 µm, Flat Tip, IC	10	2.6	445
503500530	Bare Fiber 400 µm, cylindrical tip, IC	10	2.6	750
503500520	Bare Fiber 600 µm, cylindrical tip, IC	10	2.6	860
503500510	Bare Fiber 1000 µm, cylindrical tip, IC	10	2.6	1400

Accessories

REF	Product
AB 5815-1	Y-Connector Pack
AB 2527	Percutaneous access needle 14 G x 25 cm
AB 5353	Percutaneous access needle 18 G x 20 cm
AB 5850	Adapter Touhy-Borst steril
410911200	Lipolas® LaserLift Beachbag
410911210	Lipolas® LaserLift After Care Kit
410911220-S	chin and neck band, open ear black, size S
410911220-M	chin and neck band, open ear black, size M
410911220-L	chin and neck band, open ear black, size L
410911220-XL	chin and neck band, open ear black, size XL

*Packing unit
The Leonardo® laser systems are classified as Class 3R and Class 4 laser products according to IEC 60825-1 / EN 60825-1. Avoid eye or skin exposure to direct or scattered radiation. Refer to the user manual for detailed safety instructions. All our here described products are MDR certified.

Contact us

To Learn More About a Whole New World of Minimally Invasive Laser Therapies

ENT

Thoracic Surgery

Aesthetics

Urology

Proctology

Phlebology


Aesthetics

Thyroid Treatment

Orthopaedics & Pain Management

Gynaecology

Orthopaedics & Pain Management

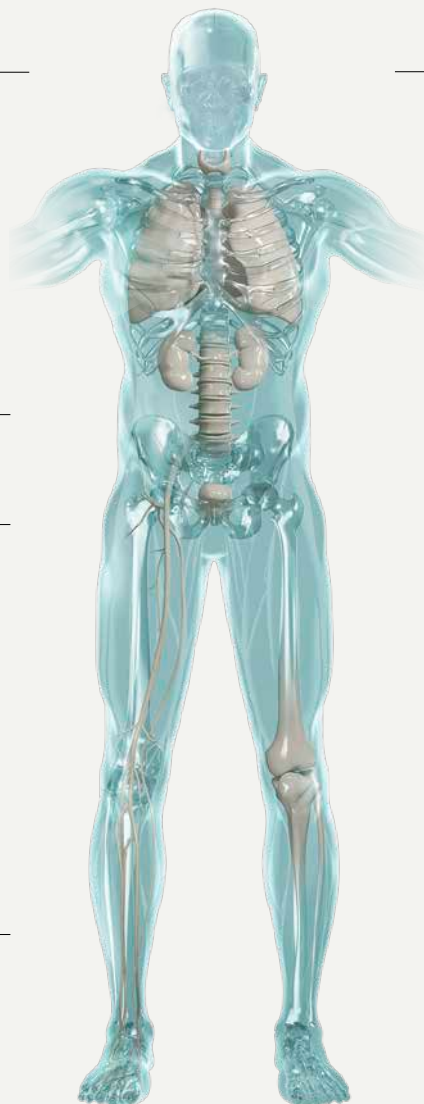

biolitec GmbH
Untere Viaduktgasse 6/9
A-1030 Wien
info@biolitec.com

Legal Manufacturer
CeramOptec GmbH
Siemensstr. 44
53121 Bonn

Find your
local biolitec®
partner

